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INFORMATION

Date of last update: 2025-02-11

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Obtain instruction from the qualified personnel in the safe use of the product.
- ▶ Please contact the qualified personnel if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

1.1 Construction and Function

Your prosthesis is a custom-made medical device. It was built and adapted to meet your personal needs and indications.

Do not modify or repair your prosthesis yourself. Attempting this can damage the prosthesis or impair its functionality. Contact your O&P professional if you have questions or in case of problems.

2 Intended use

2.1 Indications for use

The product is intended exclusively for lower limb exoprosthetic fittings.

2.2 Operating conditions

Your personal situation (e.g. training level, amputation conditions) determines your performance when using your prosthesis. The technically possible area of application depends on the components used for your prosthesis.

3 General safety instructions

CAUTION! Risk of injury and risk of product damage

- ▶ To avoid the risk of pinching, do not reach into the joint mechanism.
- ▶ Keep the product away from open flames, embers and other sources of heat.

NOTICE! Danger of product damage and restricted functionality

- ▶ Do not expose the product to prohibited environmental conditions (see instructions for use of the individual components).
- ▶ Check the product for damage if it has been exposed to prohibited environmental conditions.
- ▶ Do not use the product if it is damaged or in a questionable condition. Clean the product if it gets dirty. If necessary, have it inspected, repaired or replaced by the manufacturer or a specialist workshop.
- ▶ Use the product carefully and as intended.
- ▶ Do not make any changes to the product.
- ▶ Check the product for damage and readiness for use prior to each use.
- ▶ Do not use the product if its functionality is restricted. Take suitable measures (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop).

4 Use

WARNING

Operation of motor vehicles

Risk of accident due to limited body function

- ▶ The product has not been approved by the manufacturer for the operation of motor vehicles.
- ▶ Observe the applicable legal regulations for the operation of motor vehicles and have your driving ability examined by an authorised test centre.

4.1 Information for use

This section contains information on the use of your prosthesis. The optimum benefit can only be obtained through only error-free handling of a prosthesis with a good fit and function. Please contact your O&P professional if you have any questions or problems.

Discuss the following topics with your doctor

- ▶ The daily duration of use for the product.
- ▶ The design of accompanying therapeutic measures (e.g. physiotherapy, occupational therapy) that are recommended for achieving optimal performance.
- ▶ Measures required to maintain or improve the functionality of your amputation area.
- ▶ The required hygiene in the amputation area.

4.2 Putting on and taking off

Putting on

- 1) **CAUTION! Do not use the prosthesis if it exhibits signs of damage or wear and tear (e.g. cracks, deformations, noises, poor fit).**
Check the prosthesis for visible damage and proper functioning before putting it on.
- 2) Put on the prosthesis as instructed by your O&P professional and verify the firm fit of the prosthesis. Please contact your O&P professional if you have any questions or problems.

Taking off

- 1) Take off the prosthesis just as your O&P professional has shown you.
- 2) Clean and care for your skin after removing the prosthesis (see page 3).
- 3) Clean your prosthesis (see page 3).

4.3 Volume fluctuations

Minor volume fluctuations may be part of normal everyday life. They influence the fit of the prosthesis and must be compensated by compression therapy (compression agents and therapeutic measures). Skin damage or other injuries may occur if the prosthesis no longer fits properly.

If you have any questions or problems, please contact your doctor or O&P professional.

- Use appropriate compression agents (e.g. residual limb compression socks, wrapping the amputation area with elastic bandages, adjustable prosthesis closures) to compensate for minor volume fluctuations.
- Avoid swelling of the residual limb after taking off your prosthesis. Use liners or residual limb compression socks for this purpose or, for example, wrap the residual limb with elastic bandages.
- Ensure that the body weight remains as constant as possible. Changes in your body weight have a direct impact on the fit of the prosthesis.
- If injuries or changes (e.g. sensory disorders) occur, contact your doctor or O&P professional.

4.4 Cleaning and Care

Skin cleaning and care

- 1) Wash the skin with pH-neutral soap.
- 2) Dry the skin thoroughly.

- 3) Check (look and feel) the skin for changes and damage such as sensory disorders. Use a mirror if you can't see everywhere.
- 4) **Optional:** Use a nourishing cream/lotion.

Cleaning the prosthesis

CAUTION

Use of unsuitable cleaning agents or disinfectants

Impairment of functionality and damage due to incorrect cleaning agents or disinfectants

- ▶ Only clean the product with the approved cleaning agents.
- ▶ Only disinfect the product with the approved disinfectants.
- ▶ Observe the instructions for cleaning and care.

The precise way in which the prosthesis may be cleaned depends on the components used. In general, the following steps are possible:

- 1) Wipe the outside of the prosthesis with a damp cloth.
- 2) Clean the inside of the prosthetic socket with a cloth and pH-neutral soap.
- 3) Remove soap residue with a damp cloth. Soap residue can cause skin irritation.
- 4) Allow the prosthesis to air dry.

4.5 Selecting shoes

Your prosthesis was built for the effective heel height of your preferred shoes. If you want to use your prosthesis with other shoes, the effective heel height of the shoes must be the same. Otherwise, the alignment of the prosthesis may need to be adjusted.

INFORMATION: Bathing prostheses are usually constructed for use without footwear.

Determining the effective heel height

- Measure the sole thickness of the heel (**a**) and forefoot (**b**)
- **Effective heel height = a - b**

- 1) Do not use the prosthesis barefoot or with shoes that have an incorrect effective heel height.
- 2) Talk to your O&P professional if you want to use your prosthesis with different shoes.

5 Maintenance

In order to avoid damage to the product, adhere to the inspection intervals recommended by your O&P professional.

6 Disposal

Return the product to the O&P professional for disposal.

7 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

7.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

8 Symbols Used

REF	Article number
------------	----------------

	Global Trade Item Number
	Lot number (10)PPPPYYYYWW PPPP – plant YYYY – year of manufacture WW – week of manufacture
	Serial number (21)YYYYWWNNNN YYYY – year of manufacture WW – week of manufacture NNN – sequential number
	Manufacturer
	Medical device
	Declaration of conformity according to the applicable European directives
	Mark of conformity with the technical regulations of Ukraine

9 Suitable products

These instructions for use are suitable for the following products.

Structure of entries

Reference number: See back page of this document for product name, manufacturer (A, B, C)

Tubes

- **2R10:** Single Axis Foot Adapter, A
- **2R17:** AXON Tube Adapter, B
- **2R19:** AXON Tube Adapter, B
- **2R2:** Tube-Adapter, short, A
- **2R20:** AXON Tube Adapter, B
- **2R3:** Tube-Adapter, long, A
- **2R33:** Single Axis Adapter, A
- **2R37:** Tube Adaptor, Short, Titan, A
- **2R38:** Tube Adaptor, Long, Titan, A
- **2R49:** Tube Adaptor, long, Aluminum, A
- **2R50:** Tube Adaptor Aluminium Short, A
- **2R51:** Single Axis Foot Adapter, A
- **2R57:** Tube Adapter, short, 34 mm, Titanium, A
- **2R58:** Tube Adapter, long, 34 mm, Titanium, A
- **2R76:** Tube-Adap, Short-34mm Steel, A
- **2R77:** Tube-Adap, Long-34mm-Steel, A
- **4G70:** Lamination Anchor, A
- **4R100:** B.K. Socket Adapter-Titanium, A
- **4R101:** Sliding adapter, A
- **4R103:** Sliding Adapter -Titanium-, A
- **4R104:** Slide Adap.W/4 Screw Conn, A
- **4R111:** Rotatable 4 prong Adapter w/adj screws, A
- **4R116:** Rotatable 4 prong Adapter with Pyramid, A
- **4R118:** Adapter Plate, A
- **4R21:** Tube Clamp Adaptor, SS, A
- **4R22:** Socket Adaptor w. Adjustment Screws, A
- **4R23:** Socket Adapt.with Pyramid, A
- **4R37:** Rotatable Socket Adapter, A
- **4R41:** Socket Adapter w. Rotation Adjustment, A
- **4R42:** Socket Adapter w. Pyramid, A
- **4R43:** Lamination Anchor, A
- **4R44:** Adapter Ring, A
- **4R50:** Pyramid Adaptor with Internal Thread, A
- **4R51:** Rotatable Sock.Adapt.Titan, A
- **4R52:** Tube Clamp Adaptor, Titan, A
- **4R54:** Sock.Adapt.W/Pyramid Titan, A

- **4R55:** Sock.Adapt.W/Adjust.Screws, A
 - **4R56:** Tube Adaptor, Angled, Titan (10 -20 -30 degrees), A
 - **4R60:** Socket Adaptor-Child, A
 - **4R63:** Socket Adapter, A
 - **4R66:** Tube Clamp Adaptor, A
 - **4R68:** Socket Adaptor-Aluminum, A
 - **4R69:** Tube Clamp Adaptor-Alum., A
 - **4R72:** Double adapter, aluminium, A
 - **4R74:** Socket Adapter - aluminium -, A
 - **4R75:** Connection Adapter, A
 - **4R76:** Double Adapter, A
 - **4R77:** Socket Adapter - titanium -, A
 - **4R78:** Double Adapter, A
 - **4R82:** Tube Clamp Adapter ø34, Titan, A
 - **4R84:** Double Adapter w/Pyramid, A
 - **4R88:** Sliding Adaptor -Titanium-, A
 - **4R89:** Socket Adaptor with Rotation Pyramid, A
 - **4R91:** Tube Clamp Adapter ø34, INOX, A
 - **4R95:** Sock.Adpt./Adjust.Screws, A
 - **4R98:** Slide Clamp-Alum, A
 - **5R1:** Socket Attachment Block, A
 - **5R6:** Socket attachment block for thermo-plast socket, A
 - **5R9:** Socket Att.Block Mod.F.Ch, A
 - **6A20:** Shuttle Lock, A
 - **6A30:** Shuttle Lock, A
 - **6A40:** MagnoFlex Lock, A
- Harmony**
- **4R147:** Harmony® P3, A
 - **4R152:** Harmony® E2, A
 - **4R153:** Adapter Plate Harmony E2, A
 - **4R180:** Harmony P4 Vacuum Pump, A
 - **4R181:** Harmony P4 HD, A
 - **4R182:** Harmony P4 modular, A
 - **4R183:** Harmony P4 HD modular, A
- Prostheses, Foot**
- **1A30:** Greissinger plus foot, A
 - **1C10:** Terion, A
 - **1C11:** Terion K2 without Footshell, A
 - **1C20:** ProSymes Prosthetic Foot, A
 - **1C30:** Trias, C
 - **1C30-1:** Trias, C
 - **1C40:** C-Walk, A
 - **1C50:** Taleo, C
 - **1C51:** Taleo Vertical Shock, C
 - **1C52:** Taleo Harmony, C
 - **1C53:** Taleo LP, C
 - **1C56:** Taleo Adjust, A
 - **1C58:** Taleo Side Flex, C
 - **1C60:** Triton, C
 - **1C61:** Triton VS, C
 - **1C62:** Triton Harmony, C
 - **1C63:** Triton LP, C
 - **1C64:** Triton HP, C
 - **1C68:** Triton Side Flex, A
 - **1C70:** Evanto, A
 - **1D10:** Dynamic foot with toes, A
 - **1D11:** Dynamic Ft,Women,20mm,Toes, A
 - **1D35:** Dynamic Plus Prosthetic Foot, Dynamic Motion Prosthetic Foot, A
 - **1E91:** Runner, A
 - **1E93:** Runner junior, A
 - **1E95:** Challenger, C
 - **1G6:** Cosmetic Light Foot with Toes, A
 - **1H38:** Single Axis Foot Only Men, A
 - **1H40:** Single Axis Foot Only Men, A
 - **1K10:** Otto Bock Dynamic Foot for children, A
 - **1K30:** SACH Foot,Child, A
 - **1M10:** Adjust, A
 - **1P9:** Pirogoff Foot, A
 - **1S101:** SACH+ Foot, A
 - **1S30:** SACH Foot,Child,Medium, A
 - **1S49:** SACH Foot, A
 - **1S66:** SACH-foot, A
 - **1S90:** SACH FOOT, A
 - **1WR95:** Foot for water resistant prothesis, A
 - **2C1:** Footshell for 1M10, A
 - **2C2:** Footshell ProSymes, A
 - **2C20:** Connection Cap for 2C1 (small), A
 - **2C3:** Footshell 1C30, A
 - **2C4:** Footshell 1C40, A
 - **2C5:** Foot Cosmetics, A
 - **2C6:** Triton Footshell, A
 - **2C7:** Bionic-Foot Footshell, A
 - **2C12:** Footshell for Terion, A
 - **2C19:** Connection Cap, A
 - **2C20:** Connection Cap for 2C1 small, A
 - **2E3:** Footshell-Child, A
 - **2K34:** Shaped Ankle Block, A
 - **2R31:** Foot Adapter-Titan-, A
 - **2R40:** Foot Adapter with thread, A
 - **2R54:** Foot Adapter, A
 - **2R8:** Foot Adapter, A
 - **2Z22:** Complete bolt assembly, A
 - **2Z500:** Sole Kit 1E90 Sprinter, A
 - **2Z501:** Sole Kit 1E90 Sprinter - with Spikes, A
 - **2Z540:** Runner Sole, A

- **4R224:** Runner junior Pyramid Adapter, rotatable, A
- **FTC-2A:** Maverick Footshell, C

Prostheses, Joint

- **3B1-3:** Genium, B
- **3B5-3:** Genium X3, B
- **3B5-4:** Genium X4, B
- **3C60:** Kenevo, B
- **3C98-3:** C-Leg, B
- **3E80:** Modular Kneejoint, monocentric with hydraulic rotation and electrically controlled stand phase, A
- **3R106:** Modular Knee Joint, pneumatic, A
- **3R106-PRO:** Modular Knee Joint, servopneumatic, A
- **3R15:** Single Axis Knee Joint, Break, A
- **3R17:** Single Axis Knee Joint, Lock, A
- **3R2:** ProCarve knee joint, A
- **3R20:** Mod. Polycentric Knee Joint, A
- **3R21:** Polycntr. Knee Joint for KD, A
- **3R23:** Polycntr. Knee Joint for KD, Lock, A
- **3R30:** Polycntr. Knee Joint for KD, Titan., A
- **3R31:** Probedo, A
- **3R32:** Polycntr. Knee for KD, Lock, Titan., A
- **3R33:** Single Axis Knee Joint, Lock, Titan., A
- **3R36:** Polycntr. Knee Joint, Titan., A
- **3R38:** Single Axis Knee Joint Children, A
- **3R39:** Knee Joint Children, Lock, A
- **3R40:** Lightweight Knee Joint, Lock, A
- **3R41:** Modular Knee Joint, A
- **3R46:** Polycntr. Knee Joint for KD, Titan., A
- **3R49:** Single Axis Knee Joint, Break Titan., A
- **3R55:** Polycntr. Hydr. Knee Joint, Titan., A
- **3R60:** Modular EBS Knee Joints, A
- **3R60-PRO:** EBS-PRO-Knee, A
- **3R62:** Pheon - Modular Knee Joint, A
- **3R66:** Polycentric Knee Joint, Children, A
- **3R67:** Children's knee joint, A
- **3R78:** Modular Knee Joint, pneumatic, A
- **3R80:** Modular Knee Joint with Rotary Hydraulic, A
- **3R85:** Dynion, A
- **3R90:** Modular knee joints, A
- **3R92:** Comfort Brake Knee (pneumatic system), A
- **3R93:** Modular Friction Brake Knee Joint with Lock, A
- **3R95:** Lightweight Hydraulic Knee, A
- **3S80:** Modular Sports Knee Joint, A

- **3WR95:** Aqua Knee, A
- **4F34:** Lock Lever, A
- **4P863:** Shield insert, B
- **7E10:** Helix3D Hip joint, A
- **7E4:** Modular Hip Joint -Free Motion-, A
- **7E5:** Modular Single Axis Hip Joint, A
- **7E7:** Modular hip joint, A
- **7E8:** Modular Hip Joint-Child, A
- **7E9:** Monozentric Hip-Joint with hydraulic control mode, A

Prostheses, Leg

- **21B3:** Neoprene AK Suspension, A
- **21B4:** Support Bandage, A
- **21B37:** Ak Suspension, A
- **21Y105:** Valve for Isny Socket, A
- **21Y12:** Threaded Valve W/Housing, A
- **21Y14:** Push Valve, A
- **21Y15:** Mag Valve, A
- **21Y21:** ClickValve, A
- **21Y45:** Flat Rubber Valve, A
- **21Y95:** Flat Rubber Valve, A
- **21Y97:** Valve for Flexible Socket, A
- **2R102:** Bk Kit-34mm-Clamp-Titanium, A
- **2R103:** Bk Kit-34mm-Clamp-Steel, A
- **2R105:** Modular Transtibial Kit, A
- **2R119:** Vacuum connector, A
- **2R120:** Modular Transtibial Kit, A
- **2R121:** Modular Transtibial Kit, A
- **2R122:** Modular Transtibial Kit, A
- **2R123:** Modular Transtibial Kit, A
- **2R124:** Modular Transtibial Kit, A
- **2R125:** Modular Transtibial Kit, A
- **2R14:** Connection Plate, A
- **2R41:** Tube-Adapter, Children, A
- **2R48:** Tube Adapter, angled, Children, A
- **2WR95:** Tube Adapter f. water resist. Prothesis, A
- **452A1:** ProSeal Ring, A
- **4R10:** Quickchange, A
- **4R11:** Quickchange, A
- **4X907:** Redycer pyramid receiver, A
- **4R110:** Lamination anchor for children, B
- **4R112:** Sliding Adapter Set, B
- **4R117:** Lamination Anchor, A
- **4R119:** Lamination Anchor with angled arm, A
- **4R140:** One-way valve, A
- **4R156:** Tube Adaptor, angular 10° / 20° / 30°, Titanium, A
- **4R170:** Spherical Sliding Adapter, A
- **4R201:** Modular Transfemoral Kit, A
- **4R204:** TF Sports Foot Adapter, A

- **4R206:** TF Test-Sportfootadapter, A
- **4R208:** Definitiv-Sportadapter TT, A
- **4R210:** Anprobe-Sportadapter TT, A
- **4R220:** Dynamic Vacuum System, A
- **4R57:** Rotation Adapter, B
- **4R73:** Eccentric Adapter -titanium-, A
- **4R85:** Torsion Adapter, B
- **4R86:** Torsion Adapter, B
- **4WR95:** Lam. Anchor f. water resist. Prothesis, A
- **4X1:** X-Soft 1.10, B
- **4X440:** C-Soft Plus 1.10, B
- **5R2:** Lamination Ring up to 150 kg, A
- **6A41:** Sliding Plate (hole plate), A
- **6A53:** Sliding Adapter, A
- **6A54:** Sliding Adapter, A
- **99B116:** Soft Touch, A

Covers

- **3R24:** Foam Cover (AK), A
- **3R48:** Foam Cover (AK), A
- **3R6:** Foam Cover (AK), A
- **3S107:** Foam Cover, A
- **4P100=7:** Genium X4 Protective cover, short, B
- **4X193-1:** Genium X3 Protective Cover, B
- **4X840:** Kenevo Protective Cover, B
- **4X860:** C-Leg Protective Cover (w/o shield), B
- **4X900:** Genium X3 Protective Cover, B
- **560X12:** bebalance+ 1.6, B
- **560X500:** AxonSoft 2.2, B
- **6R18:** Cosmetic foam cover, A
- **3S106:** Pre-Shaped, Cored, Flexed AK Cover, A
- **3S124:** Pre-Shaped, Cored, Flexed AK Cover, A
- **3S36:** Cosmetic Foam Cover, A
- **3S27:** Cosmetic foam cover for hip ex., A

Prosthetic Liners

- **5A60:** Varos, B
- **6Y100:** AeroLink Hybrid Liner, A
- **6Y110:** Skeo Sealing, A
- **6Y200:** Varos Liner, A
- **6Y40:** Otto Bock Silicon Liner, A
- **6Y41:** Skeo Pure Cushion (TT), A
- **6Y42:** Skeo (TT), A
- **6Y43:** Skeo Pure Locking (TT), A
- **6Y44:** Compression Liner Cushion, A
- **6Y45:** Compression Liner Locking, A
- **6Y60:** Uneo (TT), A
- **6Y70:** Silicon Gel Liner, A
- **6Y75:** Skeo Skinguard (TT), A

- **6Y77:** SKEO 3D TT, A
- **6Y78:** Skeo 3D (TT), A
- **6Y80:** Skeo (TF), A
- **6Y81:** ProSeal SIL Liner (TF), A
- **6Y85:** Skeo Skinguard (TF), A
- **6Y87:** Skeo 3D, TF, A
- **6Y88:** SKEO 3D (TF), A
- **6Y90:** Copolymer liner with distal attachment, A
- **6Y92:** Caleo (TT), A
- **6Y93:** Caleo 3D (TT), A
- **6Y94:** Liner Dynamic Vacuum System (TT), A
- **6Y95:** Caleo 3D, A
- **452K6:** Knee Cap, reinforced, A
- **452K7:** Two-Directional Rubber Knee Cap, A
- **453A2:** Derma Protection Knee Comfort, A
- **453A3:** Derma ProFlex, A
- **453A30:** ProFlex Plus, A
- **453A4:** Derma ProFlex short, A
- **453A40:** ProFlex Plus short, A
- **454A11:** Gaitor Sleeve, C

Socks, Stump

- **451F10:** BK Prosthetic Shrinker, A
- **451F11:** AK Prosthetic Shrinker, A
- **451F12:** Compression Shrinker f.transfem.amp.CCL1, A
- **451F13:** Compression Shrinker f.transib.amp.CCL1, A
- **451F24:** Prosthetic Sock dis. Cotton-prox. Terry, A
- **451F25:** Prosthetic Sock dis. Terry-prox. Cotton, A
- **451F26:** Prosthetic Sock dis. Terry-prox. Cotton, A
- **451F27:** Prosthetic Sock dis. Cotton-prox. Terry, A
- **451F2:** Prosthetic Sock Terry, A
- **451F3:** Prosthetic Sock Cotton, A
- **451F4:** Prosthetic Sock Nylon with hole, A
- **451F6:** Special Pros.Sock for Liner, Terry Cloth, A
- **451U1:** Prosthetic Sock (BK), Wool, A
- **453D2:** Derma Seal Trans Ped, A
- **453D4:** Derma Seal forte, A
- **453D5:** Derma Seal Double forte, A
- **453D7:** Derma Seal Gel Socks, A

Battery Chargers

- **4E50-3:** Battery Charger for C-Leg, B
- **4E60:** Inductive Charger, B

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